

Validation of a multi-modal blood-based test for the detection of colorectal cancer with sub single molecule sensitivity


Kevin D'Auria, Yunpei Chang, Michael Multhaup, Haley Axelrod, Lu Zheng, Jordan Burke, William Young Greenwald, Yupeng He, Kathryn Pendleton, Nathan McDonald, Victoria M. Raymond, Anna Hartwig, Sven Duenwald, Darya Chudova, AmirAli Talasaz
Guardant Health, Redwood City, CA;



Introduction


- Blood-based colorectal cancer (CRC) screening tests can improve adherence to screening guidelines.
- Yet, current commercially available options have poor sensitivity and specificity preventing effective implementation into routine clinical care.
- Here we report the analytical and clinical validation of a blood-based test, Shield™, for the detection of CRC and advanced colorectal neoplasia (ACN); Figure 1.

Figure 1: The blood-based test, Shield, aims to detect ACN by identifying tumor-associated biomarkers including genomic and epigenomic (methylation and fragmentomics) signatures in cell-free DNA (cfDNA) and protein expression.

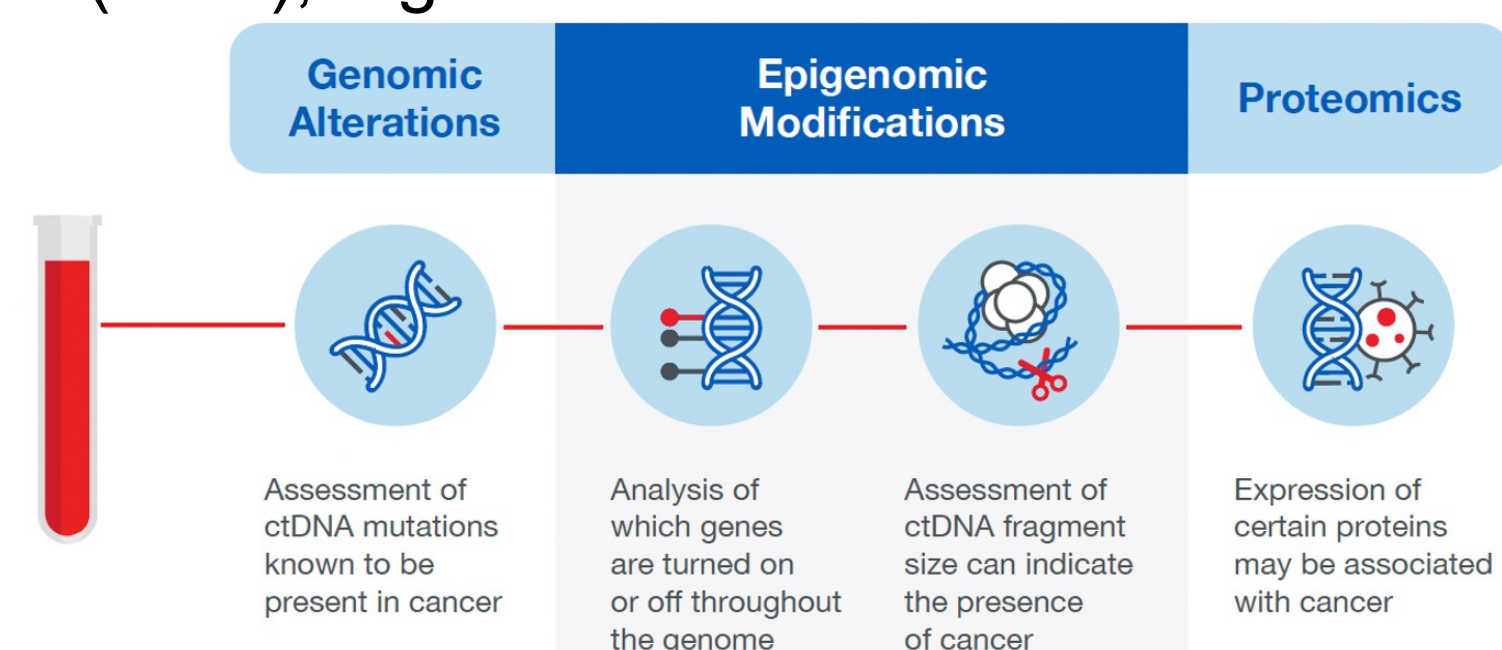

Assay workflow: Plasma-derived cfDNA was profiled using a custom assay that enriches fragments with dense CpG methylation and further depletes uninformative background molecules containing unmethylated CpGs (Figure 2). cfDNA and protein results are integrated into a binary “abnormal” vs “normal” result using a proprietary bioinformatic pipeline (Guardant Health, USA)

Figure 2: A targeted assay was designed that detects genomic, epigenomic, and proteomic modifications associated with CRC. Total cfDNA was extracted, partitioned based on methylation level, and analyzed. Data were filtered using a variant classifier to differentiate tumor derived from non-tumor derived alterations without a *priori* knowledge of tissue or germline sequencing results. cfDNA and protein results are integrated into a binary “abnormal” versus “normal” results

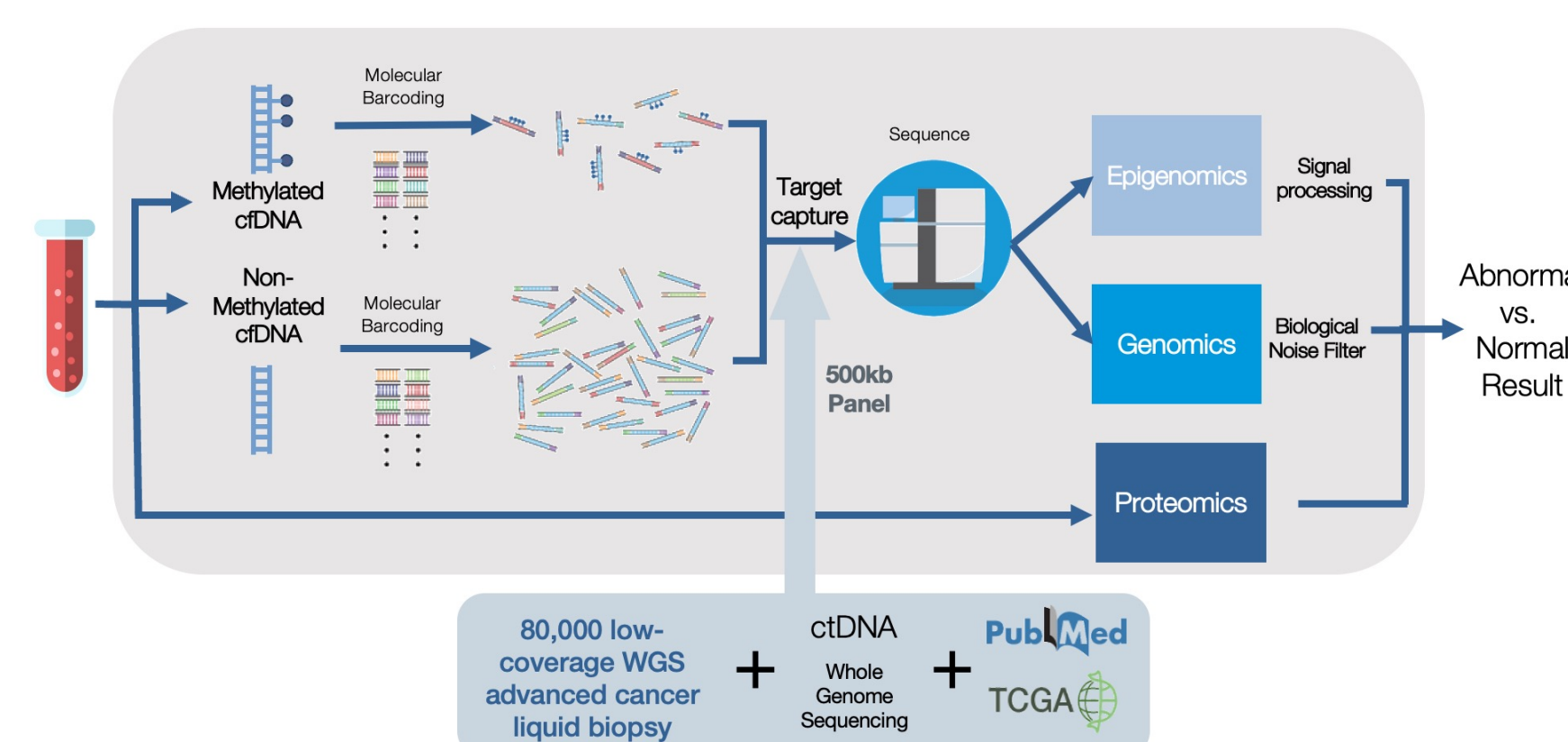

Methods

Assay Development: The assay was trained on samples from > 6,000 unique individuals

- Training: 2,685 ACN-negative and 1,698 with ACN
- Threshold Setting: 1,072 ACN-negative and 551 with ACN
- Calling thresholds were frozen prior to validation, targeting a 91.5% specificity

Analytical and Clinical Validation: Each aspect of the analytical and clinical validation study followed Nex-StoCT CLIA working group and CLSI guidelines.

- Assay was validated using samples collected from multiple cohorts, including prospective screening collections designed to capture intended use population and retrospective case-control cohorts designed to enrich for subjects with CRC.
 - CRC subjects recruited across 6 unique cohorts in US, Canada, and EU
 - Advanced adenoma (AA) and ACN negative subjects were collected in the US
- Cases and controls were balanced by age and gender


Results

Analytical Validation:

- Limit of detection (LoD) was established across six dilutions; 0.5%, 0.3%, 0.1%, 0.05%, 0.02%, and 0.01% with at least 20 replicates per dilution for CRC cases and normal controls
 - Even for low cfDNA mass inputs of less than 4ng, the 95% LoD was determined to be less than 1 tumor-derived genomic equivalent (0.5), indicating over at least 10-fold increase in assay sensitivity compared to best-in-class assays for somatic mutation detection
- Precision studies in 60 positive and 60 negative clinical sample replicates yielded > 95% average positive and negative percent agreement both within and between batches
- Endogenous interference studies in clinical positive and negative samples and minimally manipulated samples yielded > 90% positive and negative percent agreement between reference control and common endogenous substances:
 - albumin, bilirubin, hemoglobin, triglycerides, and genomic DNA

Clinical Validation:

	CRC	AA	ACN-Negative
Number of Unique Subjects	92	51	166
Median age in years (range)	64 (23 – 84)	66 (58 – 79)	64 (46 – 84)
Percent Female	46%	42%	47%

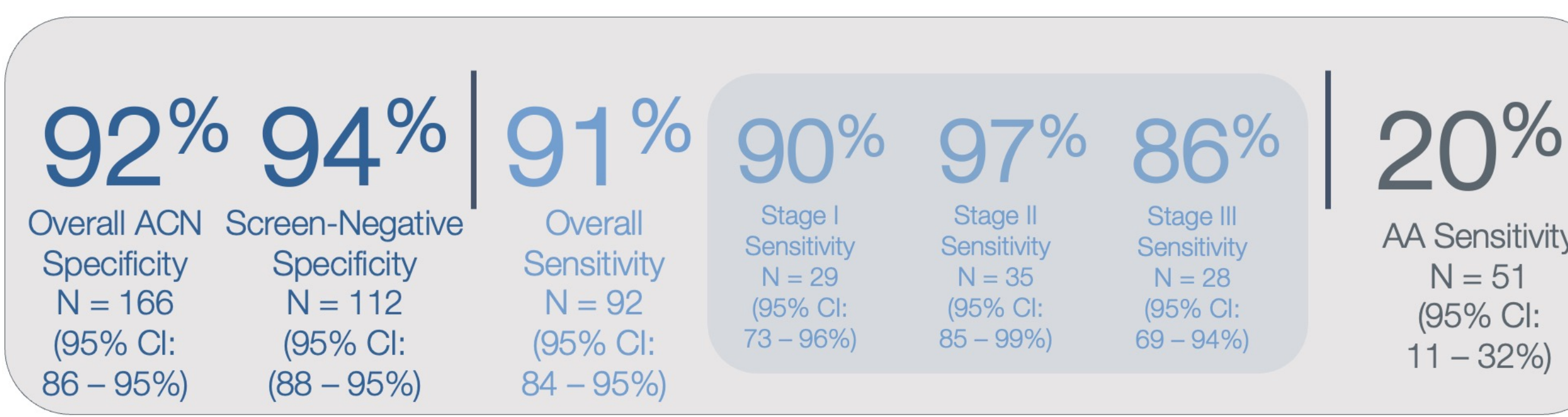

Conclusions

- Here we present the analytical and clinical validation of a multi-modal blood-based test for the detection of colorectal cancer
- Analytical validation yielded a 95% LoD at less than 1 tumor derived genomic equivalent, indicating at least a 10-fold increase in sensitivity over somatic assays
- Clinical validation demonstrated an overall sensitivity of 91% for CRC, 20% for AA, and 92% specificity for ACN with greater than 90% sensitivity for Stage I and Stage II CRC
- This test is currently being evaluated in a registrational study: **E**valuation of the **ctDNA LUNAR Test** in an **A**verage **P**atient **S**creening **E**pisode (ECLIPSE), NCT04136002
- Interventional clinical studies are in development